



BrailleNote Touch Knowledge Levels

Connecting the Brailenote Touch plus to a windows 10 computer/laptop via casting

If you are using a laptop/computer with Windows 10, you can wirelessly connect the BrailleNote Touch plus to your computer/laptop via the casting app.

To connect to your BNTP to your computer/laptop, follow these steps:

Tap on the **start menu icon** (bottom left corner) on your computer/laptop, then type "connect" in the search field. In the pop-up menu select "**Connect app**". A message on the next screen will then say that you are ready to connect.

On the BNTP if you are sighted you can turn **Keysoft off** by holding down the volume keys. Once Keysoft is off you can drag your finger downwards on the top right-hand side of the screen, then on the drop-down menu select "**cast**". On the next screen choose the relevant connection (name of your computer/laptop). Your BNTP will then attempt to connect.

Alternatively, you can access these settings on you BNTP by going to the **carousel menu** and pressing **space with O**. Once you are in the Options menu press **A** and enter to go to **Android settings**. Use your far-right thumb key to scroll down to Connected devices. Select '**cast**', then on the next screen choose the relevant connection. Your BNTP will then attempt to connect.

To **disconnect** your BNTP from your computer or laptop, simply go back to Android settings, then to **connected devices**, and select the chosen device and press disconnect. Press back to return to the carousel menu.